# ASATEX<sup>®</sup>

Art. | FMP1, FMP2, FMP2KD, FMP20,  
FMP3S, FMP3V, SMP2S, SMP3S

PPE CAT. III

EN 149:2001 + A1:2009






CE 0161

MM.YYYY

Lot:.....

- |                                    |                                   |
|------------------------------------|-----------------------------------|
| (DE) Informationen des Herstellers | (HU) A gyártó tájékoztatása       |
| (EN) Manufacturer's Information    | (IT) Informazioni del produttore  |
| (ES) Información del fabricante    | (LT) Gamintojo informacija        |
| (FR) Informations du fabricant     | (LV) Ražotāja informācija         |
| (NL) Informatie van de fabrikant   | (NO) Informasjon fra produsenten  |
| (PT) Informação do fabricante      | (RO) Informațiiile producătorului |
| (PL) Informacja producenta         | (UA) Інформація від виробника     |
| (BG) Информация на производителя   | (SI) Informacije proizvajalca     |
| (CZ) Informace výrobce             | (SK) Informácia výrobcu           |
| (DK) Informationer fra producenten | (TR) Üretici bilgileri            |
| (EE) Tootja teave                  | (SE) Tillverkarens informationer  |
| (FI) Valmistajan tiedot            | (HR) Informacije proizvođača      |
| (GR) Πληροφορίες του κατασκευαστή  |                                   |

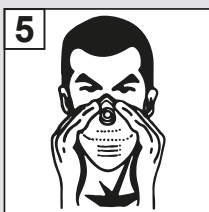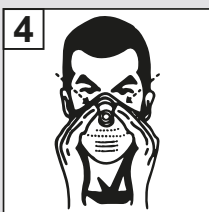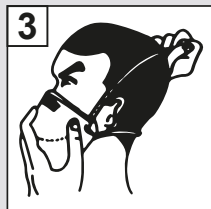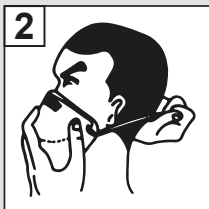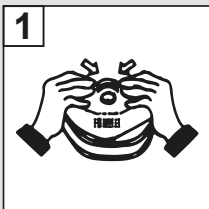**UK  
CA**

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according to Regulation (EU) 2016/425, Annex II, Section 1.4. (published in the Official Journal of the European Union). Please read carefully before using! You are required to enclose this information leaflet when passing on the personal protective equipment (PPE), or to present it personally to the recipient. You may therefore reproduce this leaflet at your own discretion.

#### Declaration of Conformity

These products meet the requirements of the Regulation (EU) 2016/425 of the European Parliament and of the council of 9 March 2016 on personal protective equipment. You can view the complete declaration of conformity at: [www.asatex.eu/konf](http://www.asatex.eu/konf)

**Manufacturer information:** ASATEX AG • August-Borsig-Str. 2 • 50126 Bergheim - Germany | **Testing institute involved:** Aitex • textile research institute • Plaza Emilio Sala, 1 • E-03801 Alcoy (Alicante). Kenn-Nr.: 0161

#### Description of all symbols and pictograms:

Indicates manufacturer information.

Maximum relative humidity (RH) during storage

End of Shelf Life

Storage temperature

Lot no. and date of manufacture: (Month/Year)

#### To select the right respirator and protection level observe national regulations.

- You should pull the respirator on, according to figures 1 – 5.
- Position nose clip above nose and squeeze the sides of the nose clip to give an exact fit with a perfect seal.
- Check fit by inhaling. If air escapes around nose, adjust position of nose clip and squeeze to fit. In case of imperfect seal elsewhere, alter fit of mask by moving straps and adapting shape of mask.
- These masks do not offer protection against gases or vapours.
- The user should be instructed how to use this equipment and should know its usage.
- The respirator must not be used in rooms with improper ventilation and of small cubage (wells, canals etc.) because of a possibility to appear deficiency of oxygen i.e. below 17% volume.
- Respiratory protection devices should be selected according to the type and concentration of hazardous substances
- Concentration and kind of contamination particles in environment should be defined.
- The masks should be provided to be used within one workshift (max 8 hours).
- In the event of difficulty in breathing, dizziness or exhaustion, leave the room immediately.
- If a user acknowledges that resistance increased considerably, the masks should be replaced with a new one.
- If there are insufficient warning properties for hazardous substances, special regulations apply for usage
- Filtering respiratory protection devices are not suitable for people with beards, sideburns or deep scars around the sealing lines
- In case of an oxygen deficiency risk, a too high concentration or hazardous substances or unknown ratios, breathing devices which are not dependent on the environmental conditions should be deployed
- The masks should be stored in closed not damaged packaging, made by the manufacturer at room temperature.
- They should not be exposed to direct sunlight.
- Provided a proper way of storage, the expiry date is indicated on the box.

Classification	Maximum penetration of the test aerosol	
	Sodium chloride test 95 l/min	Paraffin oil test 95 l/min
	% max.	% max.
FFP1	20	20
FFP2	6	6
FFP3	1	1

Classification	Maximum resistance permitted in mbar.		
	Inhalation		Exhalation
	30 l/min	95 l/min	160 l/min
FFP1	0,6	2,1	3,0
FFP2	0,7	2,4	3,0
FFP3	1,0	3,0	3,0

	FFP1	FFP2	FFP3
Filter performance	≥ 80 %	≥ 94 %	≥ 99 %
Total leakage	≤ 22 %	≤ 8 %	≤ 2 %

**FFP1** - Protection against non-toxic water- and oil-based particles, not suitable against carcinogenic substances, radioactive particles, airborne biological agents of risk groups 2 + 3 and enzymes.

**FFP2** - Protection against harmful water and oil-based particles, not suitable against carcinogenic substances, radioactive particles, airborne biological agents of risk group 3 and enzymes.

**FFP3** - Protection against harmful particles and carcinogenic particles based on water and oil, suitable against radioactive particles, airborne biological agents of risk group 2 + 3 and enzymes.

**Designations:** R = Reusable | NR = Non reusable (single shift use only)

D = Dolomite clogging test

Modell	Classification EN 149 + A1	Exhalation valve	
		with	without
FMP1	FFP1 NR		◆
FMP2, FMP2KD	FFP2 NR D	◆	
FMP2O	FFP2 NR D		◆
FMP3S, FMP3V	FFP3 NR D	◆	
SMP2S	FFP2 NR D	◆	
SMP3S	FFP3 NR D	◆	

The manufacturer was examined under the system of ensuring EU Quality of production by means of monitoring EU Regulation 2016/425 Module C2 • Aitex textile research institute • Plaza Emilio Sala, 1 • E-03801 Alcoy (Alicante). Notified body nr: 0161  
For additional technical information, please visit: [www.asatex.eu](http://www.asatex.eu)

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